



CHAIRSIDE®

Soft Reline Material

LONG-LASTING COMFORT WITH
CHAIRSIDE® SOFT RELINE MATERIAL



RELINING CHAIR SIDE? - TRY CHAIRSIDE® SOFT RELINE MATERIAL

CHAIRSIDE® Soft Reline Material is a soft, silicone based reline material for patients that require relief for full and partial maxillary and mandibular dentures.

Ideal for immediate dentures following extractions, surgery, or implant procedures, CHAIRSIDE Soft Reline Material is the material of choice when patient comfort is paramount.




PATIENT COMFORT

Silicone based material is tasteless, odorless and provides a secure long-lasting smooth finish that prevents contamination and odor. Soft silicone material is comfortable and does not harden over time.

EASY HANDLING

Auto dispensing cartridge uses up to 40% less material*, allows for quick material preparation and has an efficient working time of up to two (2) minutes with a quick five (5) minute intraoral setting time.

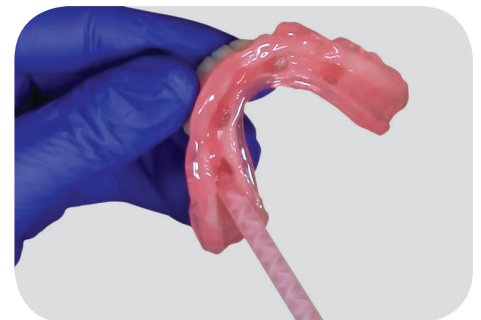


LONG LASTING

Strong denture adhesion and high tear strength provide a reline with long lasting stain resistance for wear up to 18 months. The image below shows the difference in stain resistance between CHAIRSIDE and a competitive product after immersion in curry and coffee.




*Compared to using longer tips


CHAIRSIDE® RELINE MATERIAL PROCEDURE

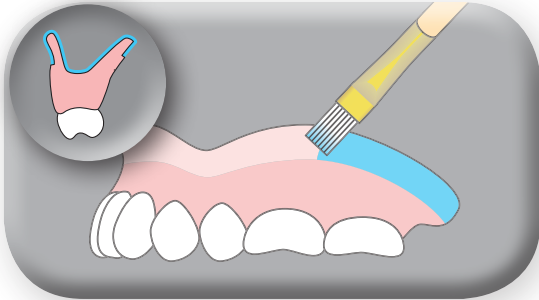


1. Apply Primer on the denture lining area, and then dry it completely.

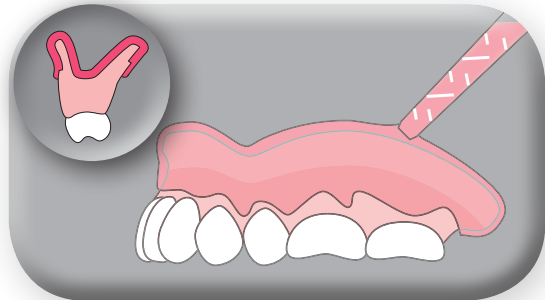


2. Apply reline material evenly across all surfaces where Primer was applied.

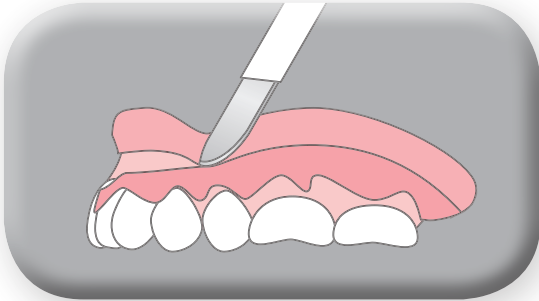


3. After intraoral setting (approximately five (5) minutes), remove the denture from mouth and carefully trim excess material with a sharp scalpel.

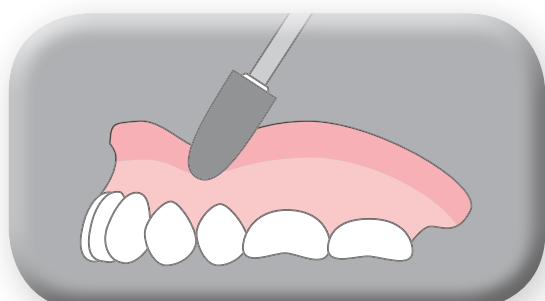

4. Use the Coarse Point (Black) to smooth the surface and adjust the shape.

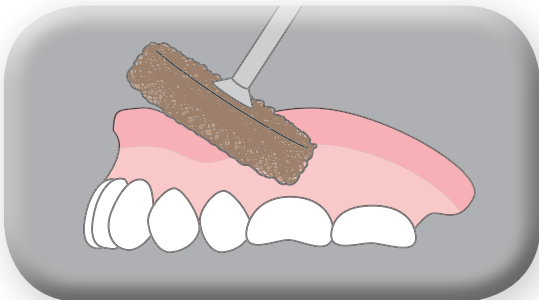

5. Use the Finishing Wheel (Brown) to finish.

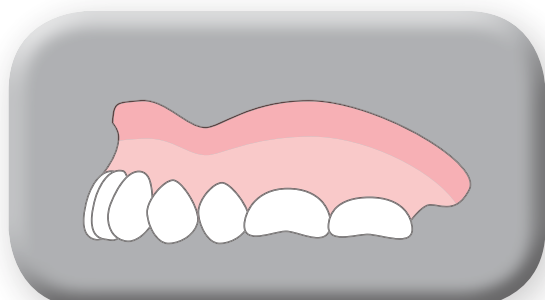

6. Reline complete.

WORKING TIMES AND MATERIAL SPECIFICATIONS

	INTRAORAL	LABORATORY
MAXIMUM WORKING TIME	Two (2) Minutes	Two (2) Minutes
SETTING TIME	Five (5) Minutes	20 Minutes
SHORE A HARDNESS	24	24
SUGGESTED USEFUL LIFE IN PATIENT	Up to 18 months	Up to 18 months



PRODUCT ORDERING INFORMATION

ITEM NUMBER	DESCRIPTION	PRICE
09573	<p>CHAIRSIDE® SOFT RELINE MATERIAL KIT CONTAINS:</p> <ul style="list-style-type: none"> • 52g Cartridge • 10mL Primer Bottle • Ten (10) Mixing Tips • Plastic Cup • Brush • Spatula • Coarse Polishing Point (Black) • Finishing Wheel (Brown) 	\$179.00



For long lasting comfort for your patient, try
CHAIRSIDE Soft Reline Material today by calling **800.262.2310**
 or visiting **www.zestdent.com**

ZD ZEST DENTAL SOLUTIONS
 ZEST | DANVILLE MATERIALS | PERIOSCOPY

ZEST DENTAL SOLUTIONS
 2875 LOKER AVENUE EAST
 CARLSBAD, CA 92010
 USA

TEL: 760.743.7744 OR 800.262.2310
 FAX: 760.743.7975
 EMAIL: SALES@ZESTDENT.COM
WWW.ZESTDENT.COM

DISTRIBUTED BY: